


Patient MP Female Age: 30 Located in Trinidad

History: This woman had poor vision from childhood but noticed a rapid deterioration since 2001. She was diagnosed with Keratoconus in 2006 and fitted with RGP lenses at that time. These were very uncomfortable and contributed to central corneal scarring and she finally discontinued wear after 6 months. As she had no option of other corneal lens types in Trinidad at that time, she had no choice but to resort to spectacles but got very poor vision with them. She was offered corneal grafts as only final option by her ophthalmologist but she could not afford the operation.


MP reported she constantly rubbed her eyes and has a history of allergies and mild asthma.

Vision: R: Hand movements L: Hand movements  
Spectacle RX: R: Plano/-3.00 x 180 VA HM L: Plano/-3.00 x 180 VA HM  
(These were the spectacles she had been wearing for 2 years)  
Auto refractor: R: -22.25/-2.50 x 20 VA 6/60 L: -20.00/-4.00 x 22 VA 6/60+1  
Keratometry: R: 6.14 x 171/5.64 x 81 L: 6.04 x 14/5.60 x 104

Topography:  
RE

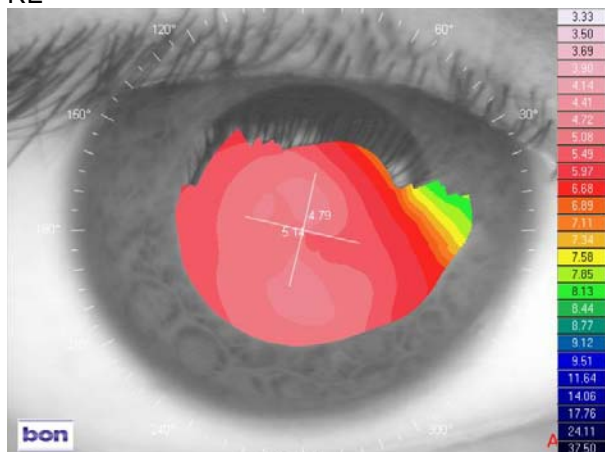

LE

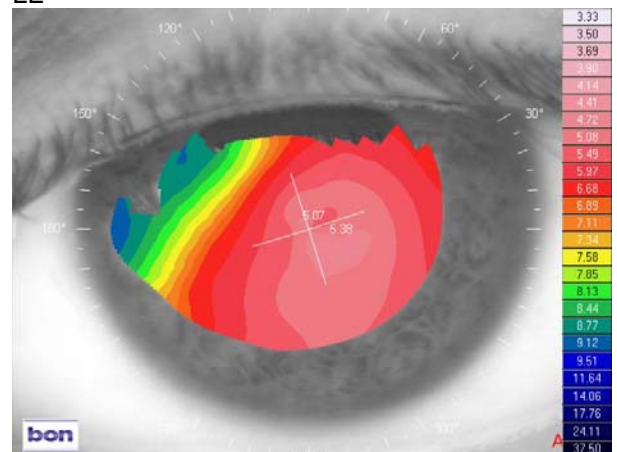

Placedo disc views

RE



LE



UK Order Line: 0800 585115 (Freephone)

Tel: +44 (0)1525 381112 Fax: +44 (0)1525 370091

Email: [orders@ultravision.co.uk](mailto:orders@ultravision.co.uk) Web: [www.ultravision.co.uk](http://www.ultravision.co.uk); [www.siliconehydrogel.co.uk](http://www.siliconehydrogel.co.uk); [www.kerasoft3.com](http://www.kerasoft3.com)

ULTRAVISION INTERNATIONAL LIMITED, COMMERCE WAY, LEIGHTON BUZZARD, BEDFORDSHIRE, LU7 4RW, UNITED KINGDOM

Lens type fitted: KeraSoft®3 by UltraVision produced in Contamac Definitive™ Silicone Hydrogel material

R: 8.20/14.00/-19.50/-4.50 x 145 VA 6/9

L: 8.20/14.00/-20.75/-4.75 x 15 VA 6/9

Wearing time: All day wear with excellent comfort.

Fitting MP with the KeraSoft®3 lenses has changed her life totally. Before, she was not even able to see the faces of her children and was incapable of earning a living. She had no prospects for improving her life as she could not afford the grafts which were the only option open to her. Although her vision is slightly limited by the central scarring caused by the RGP lenses, she is completely happy with her vision and there is no reason to proceed with a graft.

Now that she has normal vision with the KeraSoft®3 lenses, she has taken up employment and can now interact with her family normally. She was previously effectively blind and now enjoys normal vision all her waking hours. She also reports that the continual itchiness she experienced has now receded and she no longer rubs her eyes at all.

UK Order Line: 0800 585115 (Freephone)

Tel: +44 (0)1525 381112 Fax: +44 (0)1525 370091

Email: [orders@ultravision.co.uk](mailto:orders@ultravision.co.uk) Web: [www.ultravision.co.uk](http://www.ultravision.co.uk); [www.siliconehydrogel.co.uk](http://www.siliconehydrogel.co.uk); [www.kerasoft3.com](http://www.kerasoft3.com)

ULTRAVISION INTERNATIONAL LIMITED, COMMERCE WAY, LEIGHTON BUZZARD, BEDFORDSHIRE, LU7 4RW, UNITED KINGDOM